

Location Code

Different locations across the world have varying gravitational pulls. This in turn affects the accuracy of weight readings on scales. By selecting the correct gravity setting on your scale according to your geographical location, you are guaranteed the most accurate weight readings. The scale's default gravity setting is Location Code A7.

See the instruction manual to set the location.

Country/Region	City	Location Code
Afghanistan	Kabul	A5
Australia	Adelaide, Canberra, Melbourne, Sydney	A5
	Brisbane, Perth	A6
Bahrain	Manama	A7
Bangladesh	Dhaka	A7
Bhutan	Thimphu	A6
Brunei	Bandar Seri Begawan	A9
Cambodia	Phnom Penh	A8
China	Beijing	A4
	Shanghai	A6
East Timor	Dili	A8
Federated States of Micronesia	Palikir	A8
Fiji	Suva	A8
Hong Kong	Hong Kong	A7
India	Jaipur, Lucknow, New Delhi	A6
	Ahmedabad, Bhopal, Kolkata	A7
	Bangalore, Chennai, Hyderabad, Mumbai	A8
Indonesia	Jakarta, Medan	A9
Iran	Tehran	A5
	Shiraz	A6
Iraq	Baghdad	A5
Israel	Jerusalem	A6
Jordan	Amman	A6
Kiribati	Tarawa	A9
Kuwait	Kuwait	A6
Laos	Vientiane	A8
Lebanon	Beirut	A5
Malaysia	Kuala Lumpur	A9
Maldives	Male	A9
Marshall Islands	Majuro	A8

Country/Region	City	Location Code
Mongolia	Ulan Bator	A3
Myanmar	Naypyidaw	A7
	Yangon	A8
Nauru	Yaren	A9
Nepal	Kathmandu	A6
New Zealand	Wellington	A4
North Korea	Pyongyang	A4
Oman	Muscat	A7
Pakistan	Islamabad	A5
	Karachi	A7
Palau	Koror, Melekeok	A8
Papua New Guinea	Port Moresby	A8
Philippines	Manila	A8
Qatar	Doha	A7
Samoa	Apia	A8
Saudi Arabia	Riyadh	A7
Singapore	Singapore	A9
Solomon Islands	Honiara	A8
South Korea	Busan, Seoul	A5
Sri Lanka	Colombo, Sri Jayawardenepura Kotte	A8
Syria	Damascus	A5
Taiwan	Taipei	A7
Thailand	Bangkok	A8
Tonga	Nuku'alofa	A7
Tuvalu	Funafuti	A8
United Arab Emirates	Abu Dhabi	A7
Vanuatu	Port Vila	A8
Vietnam	Hanoi	A7
Yemen	Sanaa	A8

